


STERILE CONTAINER SYSTEMS



Window For Emulating Indicator

WINDOW FOR EMULATING INDICATOR

Picture 1

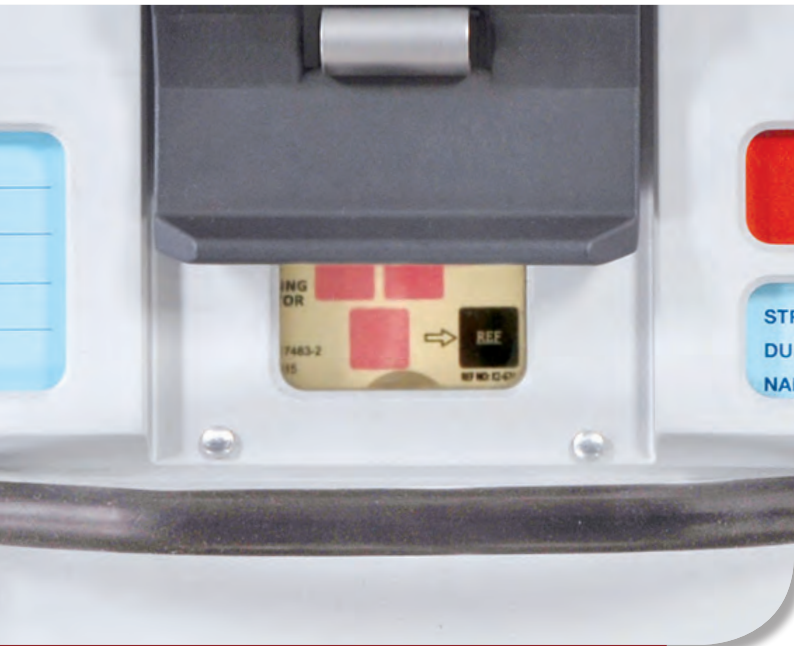

Patent Pending

We continue to produce solution for the sterilization management requirements with our 25 years of experience in medical industry.

For this purpose, we have developed emulating indicator window for the sterile container systems.



AIM OF THE WINDOW FOR EMULATING INDICATOR

Emulating indicators are used for the load sterilization monitorization while surgical instruments are loaded to containers in central sterilization services department (CSSD). (Picture 2)

During standard container usage it is not possible to see indicator results without opening the lid. When set is decided to use in surgery if result is negative and also if there is no alternative set, surgery is delaying and problems occurs in implementations.

Picture 2



EMULATING INDICATOR

K2-674 Paper 250 Pcs.

Result of the emulating indicator can observed after cycle easily from the window on the container case. In negative conditions, sterilization processes are repeated before send set to usage. By this way waste of time prohibited and it can help to avoid troubles before start.

NO RISK...

WINDOW FOR EMULATING INDICATOR

- ▶ Transparent PPSU polymer material is used on window part of the system which is suitable to autoclave. PPSU polymer material has resistance to scoring and impact also it is a special material which has heat resistance up to 330°
- ▶ Silicon joint is used between window and box to avoid sterilization risk inside the container and security is provided %100 by 4 pieces rivet connection.
- ▶ All other parts used in the system is manufactured from anodized aluminum and stainless steel.
- ▶ Indicator is designed to place the box from two directions. Also there is a spring mechanism to immobilize indicator because of the pressure inside the indicator placed sections. This spring mechanism and the immobilization property from the two side of indicator are provided to use different branded indicators to use in this system together.



USAGE OF WINDOW FOR EMULATING INDICATOR

- 1 Emulating indicator before placing materials to sterilize inside the box for the sterilization purpose is placed inside the box as below. (Picture 3)
- 2 When you placed indicator band inside the box, you should be sure that placed references and indicator should appears. You should specify the suitable position by reciprocating motions or rotating reverse the indicator band.
- 3 After you placed indicator band fairly, you can start sterilization process by placing materials to sterilize after closing the lid.



Picture 3

STANDARD MODEL
1BIO-BARRIER MODEL
33Polymers PPSU Lids
65WINDOW MODEL
85B₃ Barrier MODEL
103for STERRAD
135Dental, Mini, Special
145BASKETS
163Autolock Mechanism Emulating Indicator
183ACCESSORIES
191