


Power System

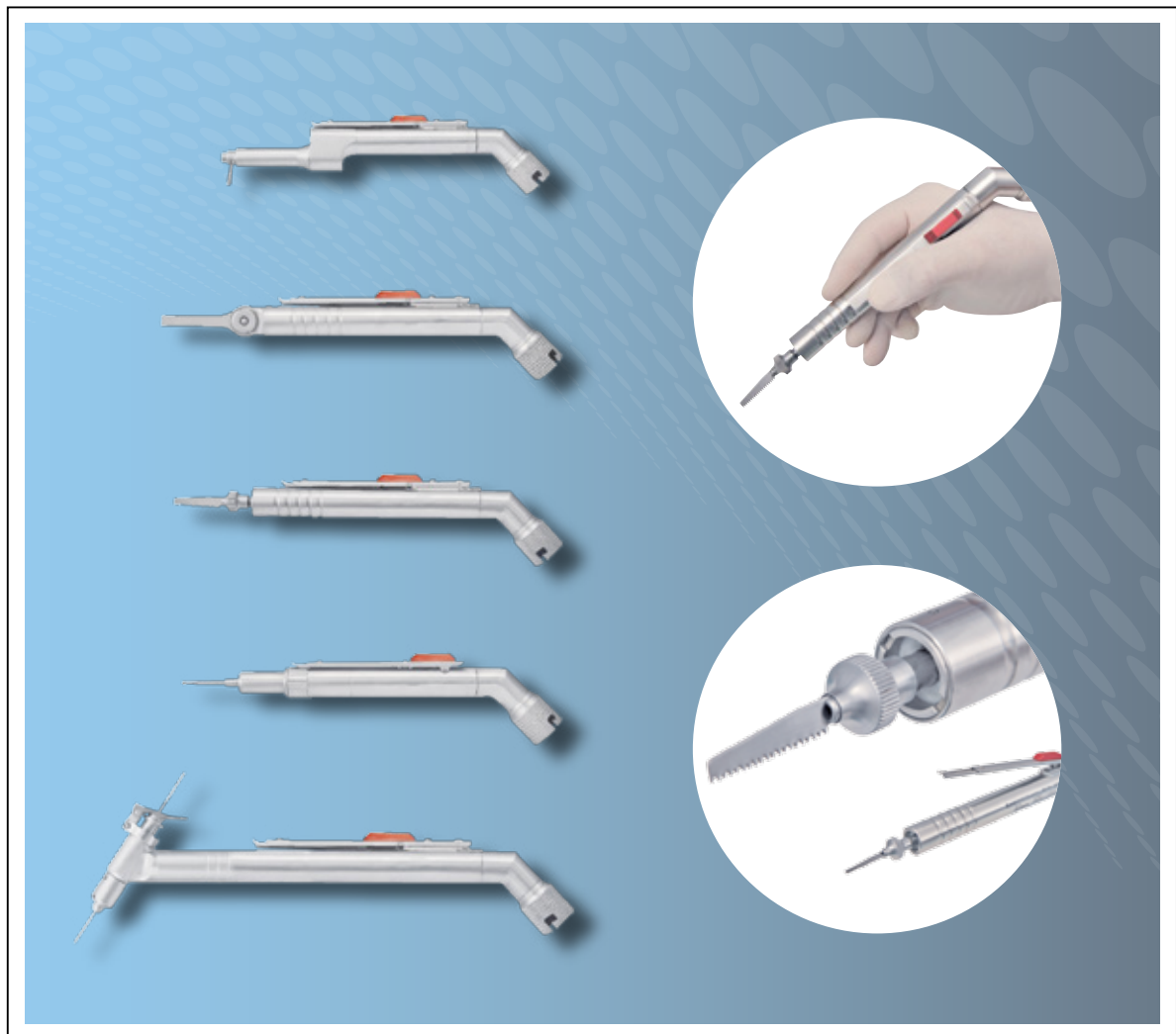

MINI COMPACT *Pneumatic Surgical Motor*

www.aygun.com



Organize Sanayi Bölgesi 55330
Kutlukent - SAMSUN / TÜRKİYE
Tel : 90 (362) 266 66 80 pbx
Fax : 90 (362) 266 93 30 pbx
E-Mail : aygun@aygun.com
Web : <http://www.aygun.com>

Edition 6

■ According to the production programme
Aygün's rights is reserved in production to improve,
modify or delete the products declared in this catalog.

KA - 4007-Y1 Copyright © Aygün Co., Inc
2013 All Right Reserved, Printed in Türkiye



AYGÜN, one of the leading establishment of Turkey in the medical industrial field, starting from neurosurgical field since 1988, at the end of its constantly R&D researches, has accomplished the production of instruments for all surgical branches. Along with meeting a large part of Turkey's requirements, it proves itself as an expert in the field with the products offered to the world market.

With present catalogues that include standard products and the special products formed on the request of the doctors including approximately 22.000 different types of instruments produced by AYGÜN. Besides being a manufacturer, it is also a designer firm.

Besides increasing its capacity, AYGÜN, in its new production plants constructed in the Organized Industrial Area of Samsun, aims to present the different models of products to the medical community.

ISO 13485:2003
CERTIFIED

CE
1984



ISO 9001:2008
CERTIFIED

www.aygun.com

AYGUN®
SURGICAL INSTRUMENTS CO., INC.


Power System



PRAXIS Battery + Electro
Surgical Motor

AYGUN®
SURGICAL INSTRUMENTS CO., INC.

Power System

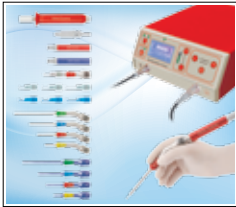


HSM High Speed Motor

90.000 rpm

AYGUN®
SURGICAL INSTRUMENTS CO., INC.


Power System



DERMATOME For Skin
Transplantation

AYGUN®
SURGICAL INSTRUMENTS CO., INC.

Power System




COMPACT Sternal Saw

Battery + Electro Surgical Motor

AYGUN®
SURGICAL INSTRUMENTS CO., INC.


Power System

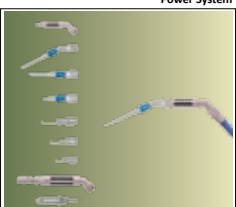

NAP Neuro Air Power
Surgical Motor

AYGUN®
SURGICAL INSTRUMENTS CO., INC.

Power System




Electro
Transmission Cable
Pneumatic

STANDARD MICRO
Surgical Motor

AYGUN®
SURGICAL INSTRUMENTS CO., INC.


Power System

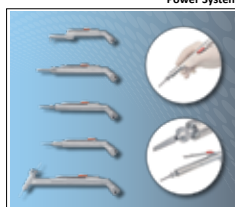

Mini COMPACT Pneumatic
Surgical Motor

AYGUN®
SURGICAL INSTRUMENTS CO., INC.

Power System

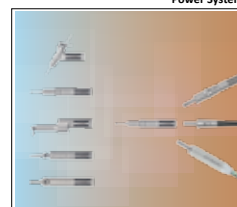

Mini Electro / Pneumatic
Surgical Motor

AYGUN®
SURGICAL INSTRUMENTS CO., INC.


Power System



STANDARD MACRO
Electro (Transmission) / Pneumatic Surgical Motor

AYGUN®
SURGICAL INSTRUMENTS CO., INC.


Power System

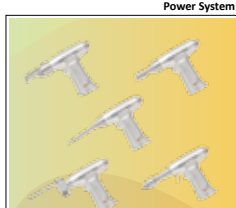

EPS Electro (Transmission)
Pneumatic Surgical Motor

AYGUN®
SURGICAL INSTRUMENTS CO., INC.


Power System

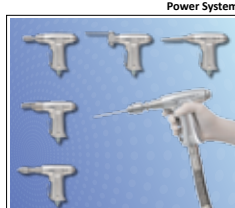


Macro COMPACT Pneumatic
Surgical Motor

AYGUN®
SURGICAL INSTRUMENTS CO., INC.

Power System



MOBIL STATOR For Surgical Motor

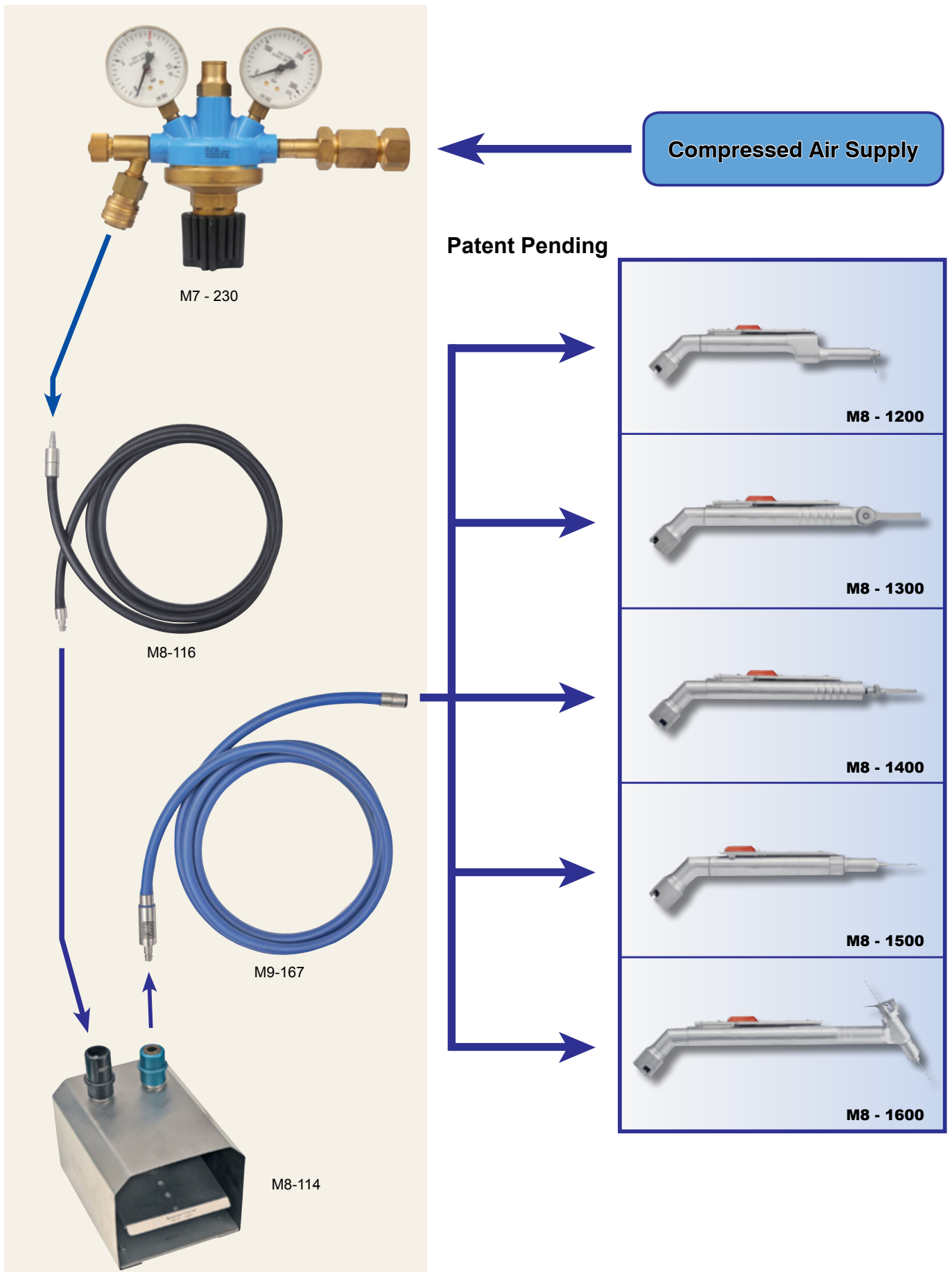




M8 - 510

Spray oil, for motor and additional pcs before sterilization.

After cleaning of every use. Oil inside of all moving pcs of device by spraying method and run the device for 2-3 seconds. Practice spraying inside of the device until the black used oil comes out and new oil clearly appears out of device as clear color and then put the device in store or process sterilization.

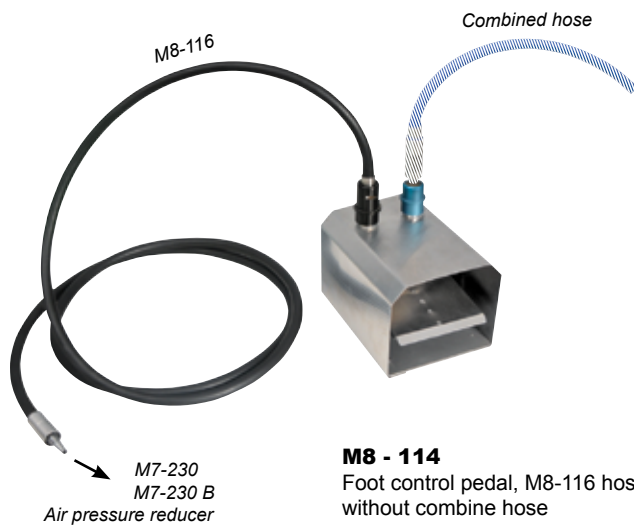

M8 - 114

Foot control pedal, M8-116 hose, without combine hose



M9 - 167 3 m

M9 - 168 5 m

Combine hoses for air supply



M7 - 230 B

Air pressure reducer, two way



M7 - 230

Air pressure reducer, one way


For security; the systems must be operated by using compressed dry air or nitrogen gas, the oxygen gas must never be used.




M9 - 167
M9 - 168



M7 - 230

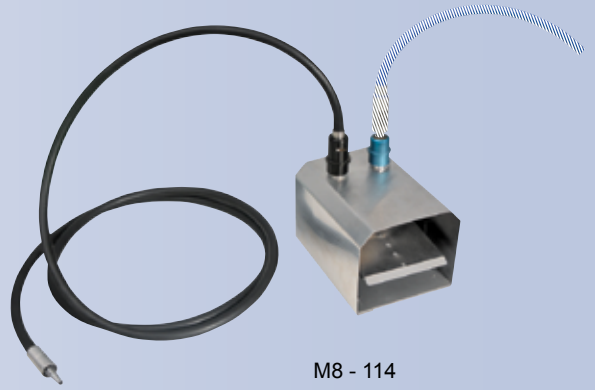



M8 - 114

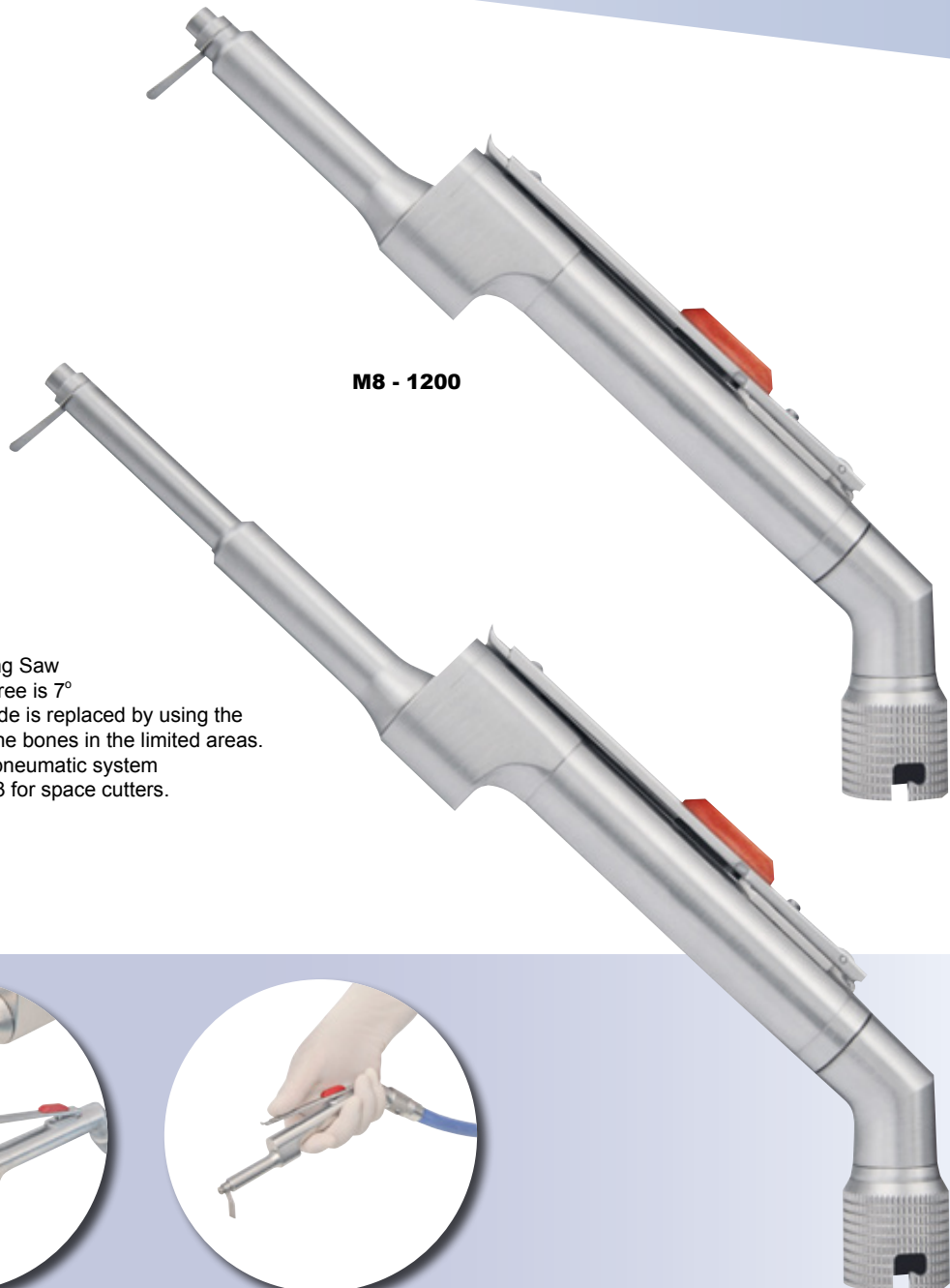

M8 - 1200

M8 - 1201

Micro Oscillating Saw
Oscillating degree is 7°
The cutting blade is replaced by using the
key in cutting the bones in the limited areas.
Only use with pneumatic system
See page No: 3 for space cutters.

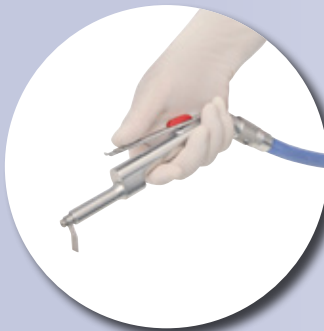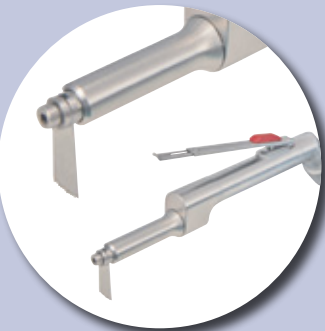

MICRO OSCILLATING SAW BLADES

M8 - 1220



M8 - 1221




M8 - 1222



M8 - 1223




M8 - 1224

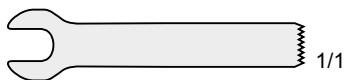


M8 - 1225




M8 - 1226

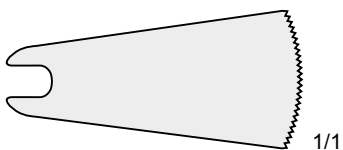


M8 - 1227

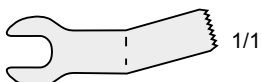


M8 - 1228

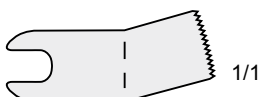


M8 - 1229

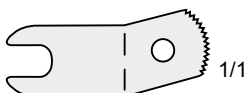


M8 - 1230

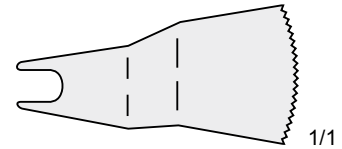

M8 - 1231

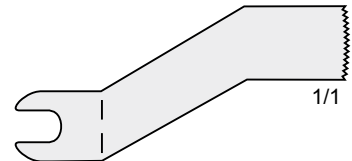

M8 - 1232

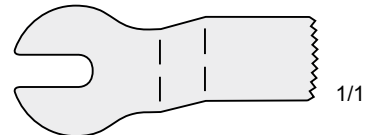


M8 - 1233




M8 - 1234



M8 - 1235

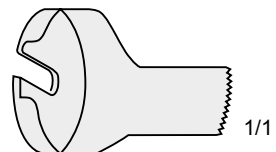

M8 - 1236

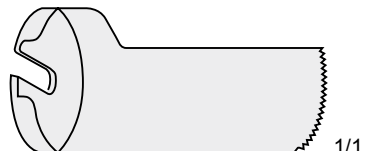

M8 - 1237






M9 - 167
M9 - 168



M7 - 230

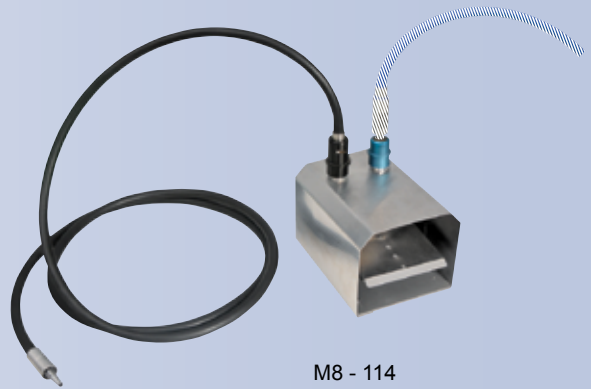

M8 - 114



M8 - 1300

Micro Sagittal Saw

It is used in deep cutting the bones and other small bones in the mandibular and maxillofacial surgery.

The cutting blade is replaced by key

Only use with pneumatic system.



(M7-320 Key included)

See page No: 5 for space cutters.




M7 - 320

Key only,
for changing
saw blades

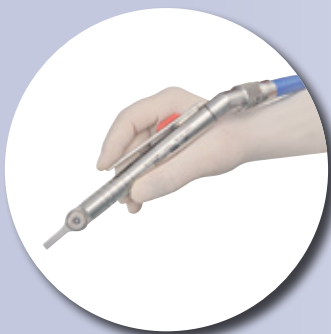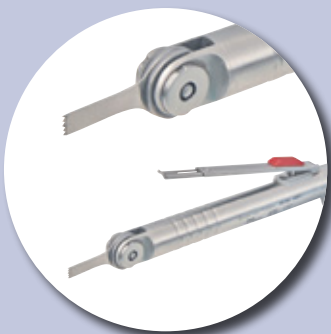


MICRO SAGITTAL SAW BLADES


M8 - 1320

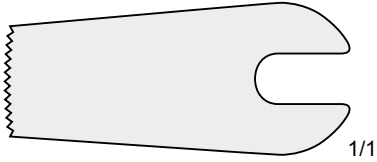

M8 - 1321

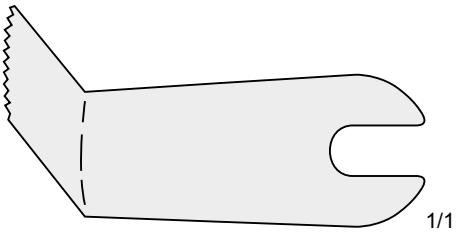


M8 - 1322

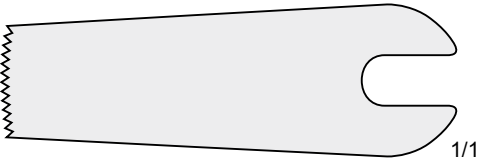


M8 - 1323




M8 - 1324

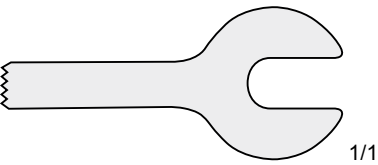


M8 - 1325

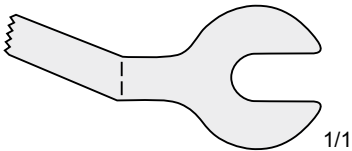


M8 - 1326

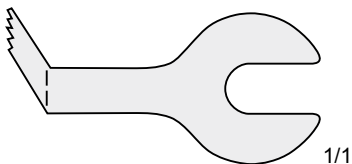


M8 - 1327

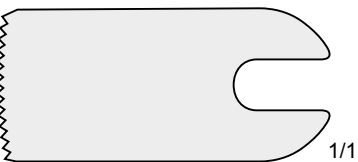


M8 - 1328

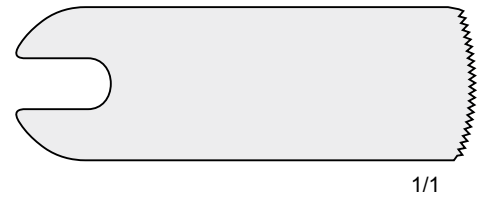


M8 - 1329

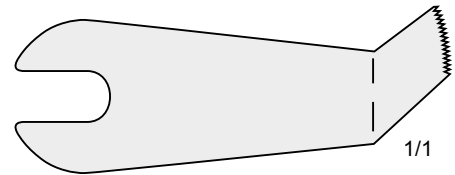


M8 - 1330

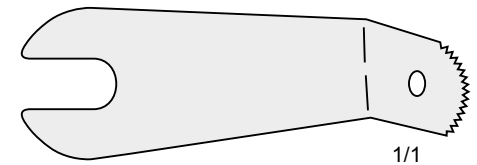


M8 - 1331

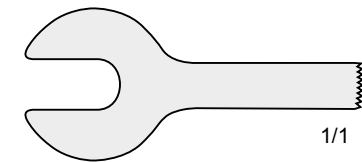


M8 - 1332

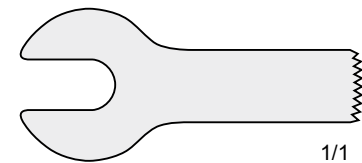

M8 - 1333

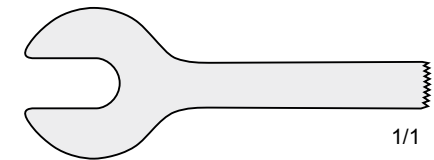

M8 - 1334

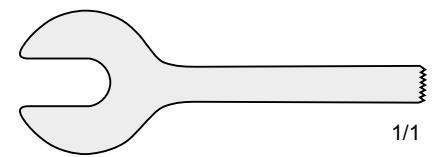





M9 - 167
M9 - 168



M7 - 230

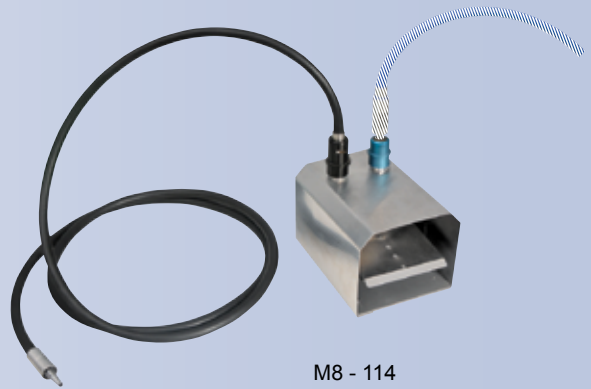

M8 - 114

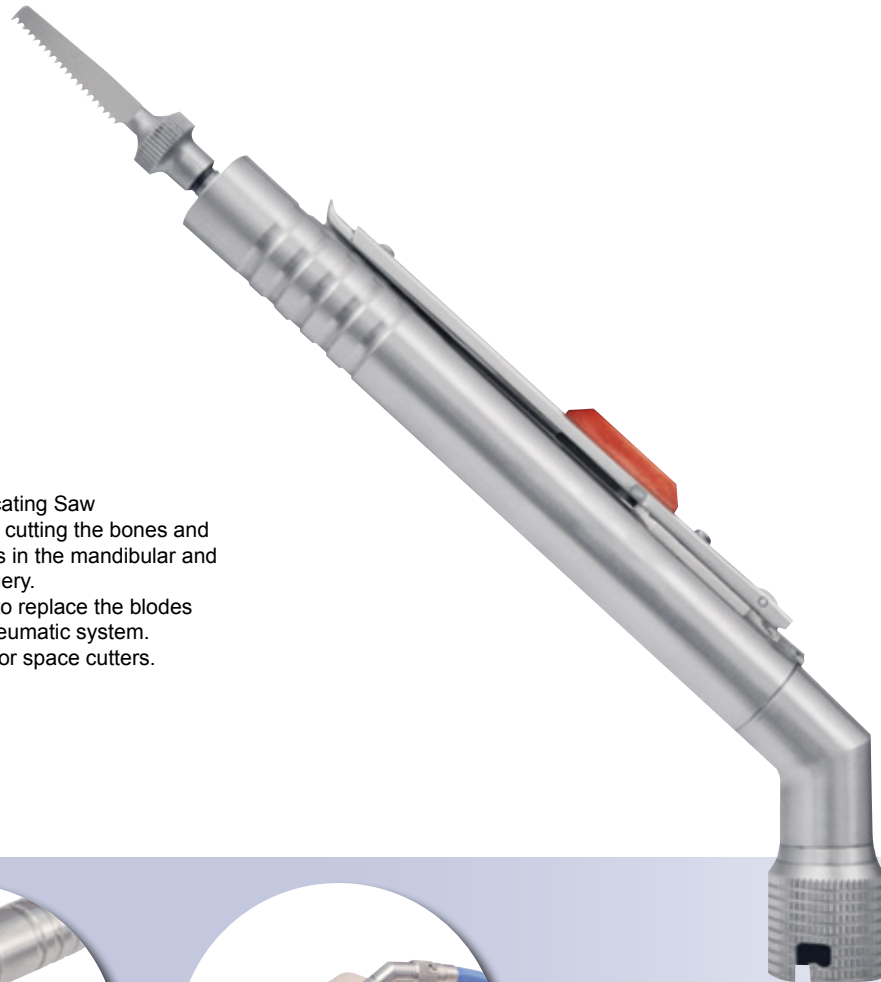

M8 - 1400

Straight Reciprocating Saw

It is used in deep cutting the bones and other small bones in the mandibular and maxillofacial surgery.


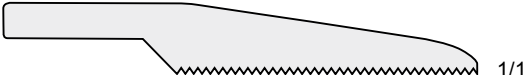









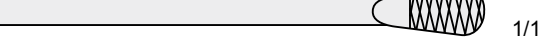

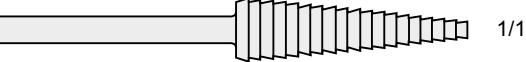
No need for key to replace the blades

Only use with pneumatic system.

See page No: 7 for space cutters.

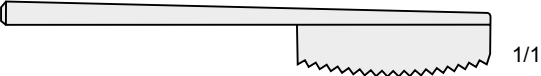

MICRO RECIPROCATING SAW BLADES

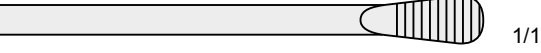
M8 - 1420	
M8 - 1421	
M8 - 1422	
M8 - 1423	
M8 - 1424	
M8 - 1425	
M8 - 1426	
M8 - 1427	
M8 - 1428	
M8 - 1429	
M8 - 1430	
M8 - 1431	
M8 - 1432	
M8 - 1433	



M9 - 167
M9 - 168



M7 - 230

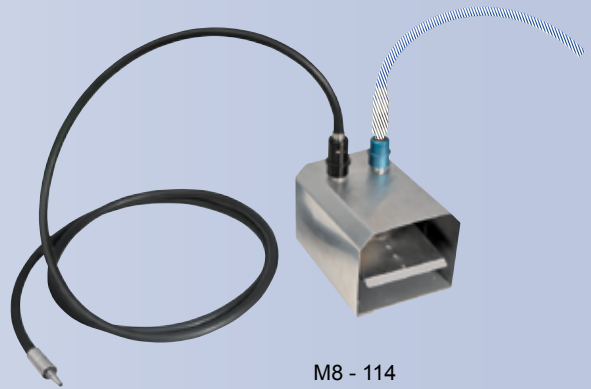

M8 - 114



M8 - 1500


Micro Drill

Micro drill has a full range, variable-speed control from 500 to 20.000 rpm.


No key is needed

Only use with pneumatic system.


See page No: 9 for space cutters.

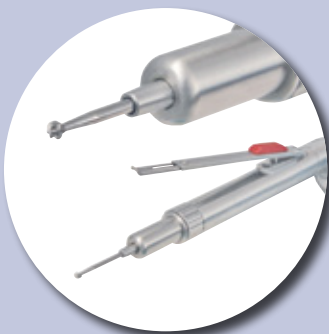


BURRS-SHANK 70 mm LONG, SHANK Ø 2.35 mm, ROUND SHANK

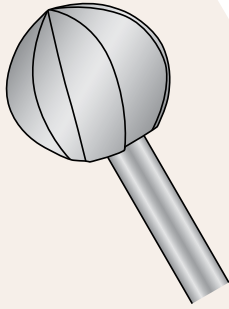



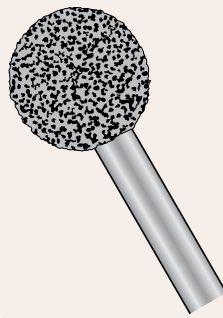



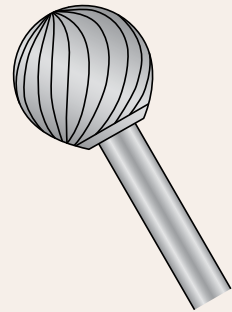







ROSEN
Stainless steel



DIAMOND



HIGH SPEED
Tungsten


											
 ROSEN	M8-1520	M8-1521	M8-1522	M8-1523	M8-1524	M8-1525	M8-1526	M8-1527	M8-1528	M8-1529	M8-1530
 DIAMOND	M8-1550	M8-1551	M8-1552	M8-1553	M8-1554	M8-1555	M8-1556	M8-1557	M8-1558	M8-1559	M8-1560
 HIGH SPEED	M8-1570	M8-1571	M8-1572	M8-1573	M8-1574	M8-1575	M8-1576	M8-1577	M8-1578	M8-1579	M8-1580



M9 - 167
M9 - 168





M7 - 230

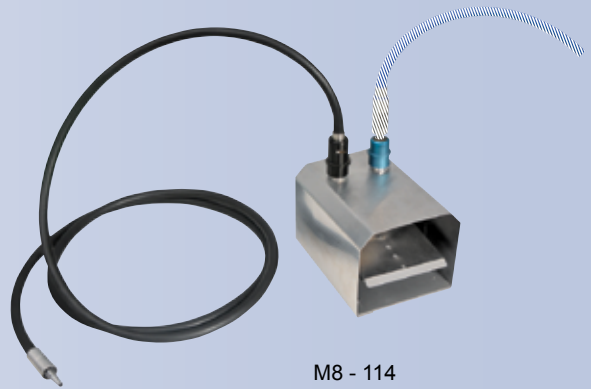


M8 - 114



M8 - 1600
K - Wire Driver
for fixating small bones
Ø 0.6 mm - 1.6 mm
Cannulated

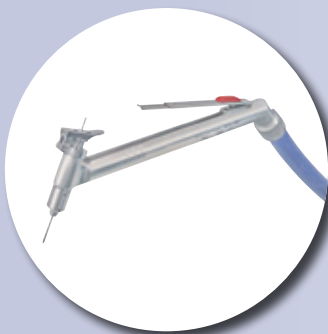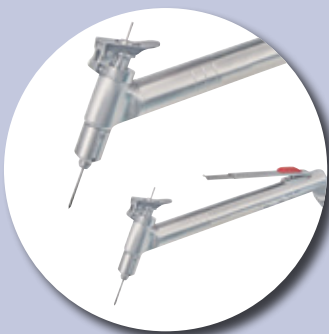

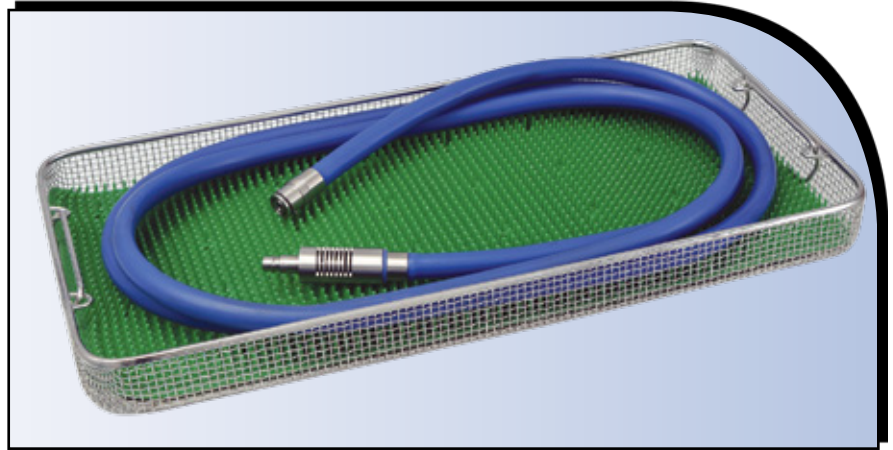

Power System

MINI COMPACT



Edition 6

Meeting Point For Surgery

AYGÜN®
SURGICAL INSTRUMENTS CO., INC.

Organize Sanayi Bölgesi 55330
Kutlukent - SAMSUN / TÜRKİYE
Tel : 90 (362) 266 66 80 pbx
Fax : 90 (362) 266 93 30 pbx
E-Mail : aygun@aygun.com
Web : <http://www.aygun.com>

www.aygun.com